

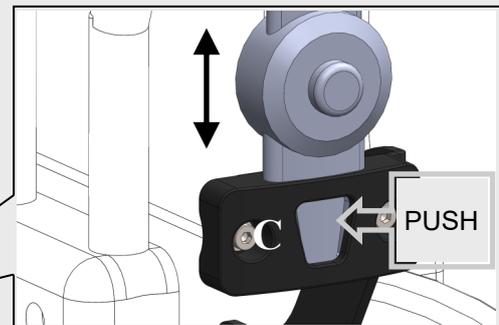


# MY Lyte Canopy

## Installation Guidelines

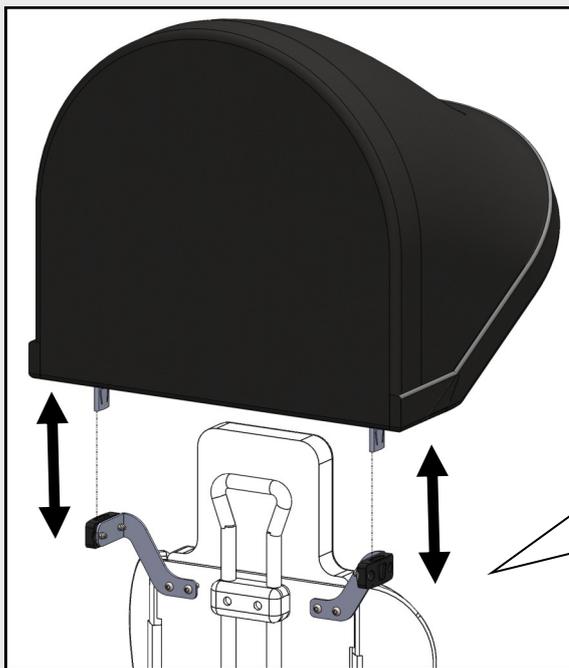

- Do not put items on the canopy when open or folded.
- Do not apply excessive force when opening and closing the canopy.
- Regularly inspect the canopy and mounts for damage and security.
- Remove Canopy before folding the MY Lyte buggy.
- 4mm allen key.

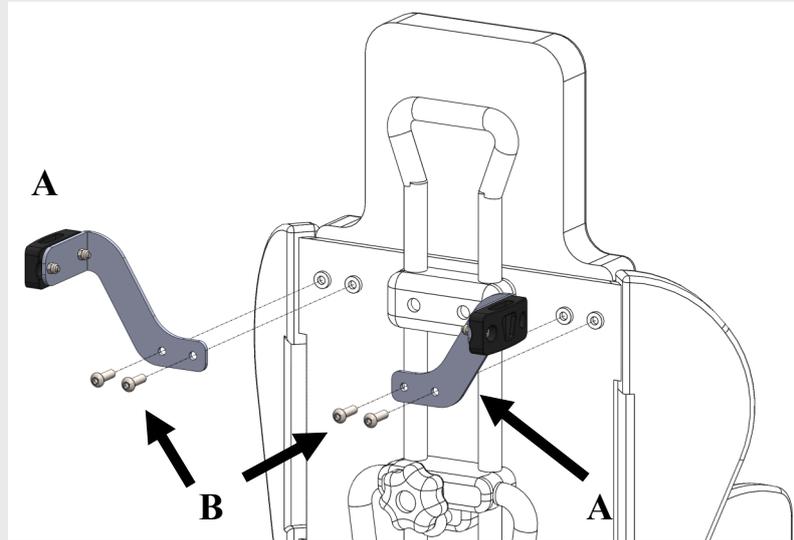

### Fitting Instructions



- Attach the mounting brackets **A** to the mounting holes at the rear of the MY Lyte back rest using the screws **B** provided.
- Insert the canopy locating arms into the mounting brackets until tab **C** clicks into place.
- Check alignment and angle of the canopy as required.
- To remove depress the centre tab **C** on the locating arm and carefully lift upwards.

# MY Lyte Rain Cover

## Installation Guidelines

Slip top of rain cover over the canopy and pull around canopy.



Bottom of rain cover slips over the footrest.



Thompson House, Unit 10 Styles Close,  
Sittingbourne, Kent, ME10 3BF  
Tel: 01795 477280 Fax: 01795 229692  
E-mail address: [sales@rms-kent.co.uk](mailto:sales@rms-kent.co.uk)  
[www.inedawheelchair.co.uk](http://www.inedawheelchair.co.uk)



FM 661078